

## 1013100 BLUESTAR FIRST AID BOX W/WATERJEL 1013300 BLUESTAR FIRST AID BOX

### DESCRIPTION

Robust plastic box containing exactly what it takes to remedy a minor accident and burns. Remember to have multiple first aid boxes located in the company so that precious time is not lost. The box comes with wall brackets of heavy plastic quality.

### VARIATIONS

- 1013100 – BlueStar First Aid Box with Waterjel
- 1013300 – BlueStar First Aid Box

### CONTAINS

- 1 x PE Plaster - 6 cm x 1 meter
- 1 x Scissor – 12,7 cm.
- 1 x Metal tweezer – 8,8 cm.
- 1 x Emergency blanket – 160 x 210 cm. (golden/silver)
- 1 x Nonwoven plaster – 10 x 6 cm. bag with 8 pcs.
- 1 x H-plaster - 4 x 7 cm. finger plasters, 10 pcs.
- 3 x Saltwater pipettes - 9,9 ml. \*\*
- 1 x Bag with 4 nitril gloves.
- 1 x Nonwoven adhesive tape roll - 2.5 cm x 5 meter.
- 1 x Triangular bandage - 136 x 96 x 96 cm.
- 1 x Sterile wound dressing - 17 x 17 cm.
- 1 x AQR aqua resist plaster – Bag with 50 pcs. various sizes.
- 1 x Skin cleansing swab, 12 pcs.
- 1 x Dry swab bag with 10 pcs.
- 2 x Elastic bandages 8 cm x 4 meter.
  
- 1 x Waterjel 10 x 10 cm. (only in **1013100**)

### REFILLS

- Complete refill with Waterjel (1013100) - Please order 1013200
- Complete refill (1013300) – Please order 1013301

\*\*Alternative item 1022500 (1 pkg. w/10 pcs.)



*Print and color may vary.*

# 1013909 BLUESTAR PE PLASTER

## DESCRIPTION

Universal plaster for use on minor cuts and abrasions.  
Protects minor wounds.

## FEATURES

- Stretchy
- Comfortable
- Waterproof
- Good air permeability



## DIRECTION FOR USE

Clean wound and surrounding area and dry skin before applying the plaster. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet.

## CAUTION FOR USE

Do not use with major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster and consult a doctor immediately. If you are diabetic consult your doctor before use.

*Print and color may vary.*

## STORAGE

Keep dry and out of direct sunlight. Store below 25 °C.

## SPECIFICATIONS

<b>Product name</b>	BlueStar PE Plaster
<b>Art.No.</b>	1013909
<b>CE Class</b>	I
<b>Material</b>	Absorbent pad: 70% viscose + 30% polyester Plaster film: PE
<b>Weight</b>	Absorbent pad: 110 gsm (per unit area) Plaster film: 60 gsm (per unit area without glue)
<b>Absorptive capacity</b>	≥600%
<b>Size</b>	6 cm x 1 meter
<b>Shelf life</b>	5 years
<b>Bioburden control</b>	Total bacteria <100 CFU/g Yeast/mouldiness ≤20 CFU/g Pathogenic: 0.
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

## 1013914 BLUESTAR SCISSOR

### DESCRIPTION

For cutting plaster, bandage, gauze, or other small first aid devices. The scissor has a rounded cutting edge in order to protect the user from self inflicted injuries.

### FEATURES

- Stainless steel

### CAUTION FOR USE

Keep out of reach of children.

### STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat



*Print and color may vary.*

### SPECIFICATIONS

<b>Product name</b>	BlueStar Scissor
<b>Art.No.</b>	1013914
<b>CE Class</b>	N/A
<b>Material</b>	Stainless steel and plastic (GPPS, HIPS)
<b>Size</b>	12,7 cm.
<b>Shelf life</b>	N/A

## 1013915 BLUESTAR TWEEZER

### DESCRIPTION

For removing splinters or other small objects in an emergency situation.

### FEATURES

- Stainless steel

### CAUTION FOR USE

Keep out of reach of children.

### STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat. Store below 30 °C.



*Print and color may vary.*

### SPECIFICATIONS

<b>Product name</b>	BlueStar Tweezer
<b>Art.No.</b>	1013915
<b>CE Class</b>	N/A
<b>Material</b>	Stainless steel
<b>Size</b>	8,8 cm.
<b>Shelf life</b>	N/A

# 1013925 BLUESTAR EMERGENCY BLANKET

## DESCRIPTION

To protect against heat and cold. The emergency blanket can provide protection against heat, cold and humidity in emergency situations.

## DIRECTION FOR USE

Emergency blanket to be used only once. Open the sachet and unfold the blanket as required. Two sides with different functions:

- The golden side absorbs heat and keeps you warm, providing a defence against potential hypothermia, maintaining your body temperature, and protecting you from the wind and humidity.
- The silver reflective outer side keeps you cool and keeps you chilled (cars, tents etc.).



## CAUTION FOR USE

Do not cover your head, danger of suffocating. Single use only. Do not use the emergency blanket during storms, near to incandescent substances or if using an electrical device (e.g. DSA). Do not cut or create holes. Store in a dry place.

*Print and color may vary.*

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Can be put into the sleeping bags.

## SPECIFICATIONS

<b>Product name</b>	BlueStar Emergency Blanket
<b>Art.No.</b>	1013925
<b>CE Class</b>	I
<b>Material</b>	PET (Polyethyleneterephthalate), Aluminium, Resin
<b>Size (unfolded)</b>	160 x 210 cm.
<b>Thickness</b>	≥1.2S (1.2S (1.2S=12µm))
<b>Shelf life</b>	5 years
<b>Tear strength</b>	≥50N
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

# 1013936 BLUESTAR NONWOVEN PLASTER

## DESCRIPTION

For use on minor cuts and abrasions. Protects minor wounds.

## FEATURES

- Good air permeability

## DIRECTION FOR USE

Clean wound and surrounding area and dry skin before applying the plaster. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet.

## CAUTION FOR USE

Do not use with major wounds. If wound bleeds excessively or skin becomes irritated or infected remove plaster and consult a doctor immediately. If you are diabetic consult your doctor before use.

## STORAGE

Keep dry and out of direct sunlight. Store below 25 °C.

## SPECIFICATIONS

<b>Product name</b>	BlueStar Nonwoven Plaster
<b>Art.No.</b>	1013936
<b>CE Class</b>	I
<b>Material</b>	Absorbent pad: 70% viscose and 30% polyester Film: Nonwoven
<b>Weight</b>	Absorbent pad: 110 gsm (per unit area)
<b>Absorptive capacity</b>	≥600%
<b>Size</b>	10 x 6 cm.
<b>Bioburden control</b>	Total bacteria <100 CFU/g Yeast/mouldiness ≤20 CFU/g Pathogenic: 0.
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016



*Print and color may vary.*

# 1013937 BLUESTAR PE FINGER PLASTER

## DESCRIPTION

Adhesive bandage for use on minor cuts and abrasions. Protects minor wounds.

## FEATURES

- Stretchy
- Comfortable
- Waterproof
- Good air permeability

## DIRECTION FOR USE

Clean wound and surrounding area and dry skin before applying plaster. Remove plaster from protective wrapper. Place pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if wet.

## CAUTION FOR USE


Use immediately after opening protective wrapper. Don't use with major wounds. Wrapper contains natural rubber latex. If wound bleeds excessively or skin becomes irritated or infected remove plaster and consult a doctor immediately. If diabetic consult your doctor before use.

## STORAGE

Keep dry and out of direct sunlight. Store below 25 °C.

## SPECIFICATIONS

<b>Product name</b>	BlueStar PE Finger Plaster
<b>Art.No.</b>	1013937
<b>CE Class</b>	I Sterile
<b>Material</b>	Absorbent pad: 70% viscose + 30% polyester. Plaster film: PEVA.
<b>Weight</b>	Absorbent pad: 110 gsm (per unit area). Plaster film: 55 gsm (per unit area without glue)
<b>Absorptive capacity</b>	Absorbent pad: ≥600%
<b>Size</b>	4 x 7 cm.
<b>Shelf life</b>	5 years
<b>Bioburden control</b>	Total bacteria <100 CFU/g Yeast/mouldiness ≤20 CFU/g Pathogenic: 0.
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

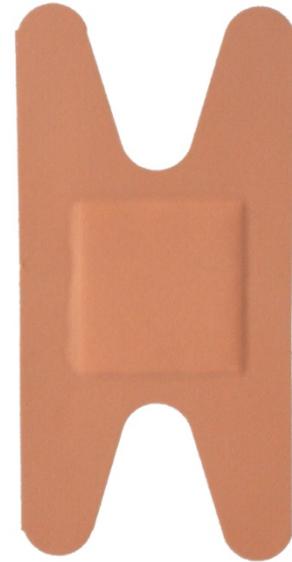

*Print and color may vary.*

# BLUESTAR SALTWATER SOLUTION

## DESCRIPTION

Sterile saline solution for emergency rinsing of a wound or flushing the eyes or nose. Sterile sodium chloride 0.9%.

## DIRECTION FOR USE

Twist the lid. Rinse the eye, wound or nose carefully.

## CAUTION FOR USE

Not for injection. Not to be taken internally. Do not use if damaged. Once opened, discard any unused solution. Use before the expiry date. Risk of infection by reuse. Storage temperature 5-30 °C.



*Print and color may vary.*

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat (store below 30 °C).

## SPECIFICATIONS

<b>Product name</b>	BlueStar Saltwater Solution
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	N/A
<b>Material</b>	Packaging: LDPE
<b>Volume</b>	15 ml.
<b>Ingredients</b>	Aqua CAS#: 7732-18-5 0.9% NaCl CAS# 7647-14-5
<b>Shelf life</b>	3 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2012
<b>Sterilization method</b>	R

# BLUESTAR GLOVES NITRIL

## DESCRIPTION

To protect against infections. Disposable. Latex-free.

## FEATURES

- Latex-free

## CAUTION FOR USE

Non-sterile. Avoid direct contact with the wound.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat.



## SPECIFICATIONS

*Print and color may vary.*

<b>Product name</b>	BlueStar Gloves Nitril
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I (non-sterile)
<b>Material</b>	Nitril without powder, NBR, Pigment
<b>Color</b>	Blue
<b>Size</b>	Large: Length: $\geq 230$ mm and Width: 110 mm
<b>Thickness</b>	$\geq 0.08$ mm and $\leq 0.22$ mm
<b>Shelf life</b>	5 years intended
<b>Certificates TÜV SUD</b>	EN ISO 13485:2012

# BLUESTAR NON WOVEN TAPE

## DESCRIPTION

Adhesive tape roll for fixing the dressings, compress, cotton ball or other small medical pads.

## DIRECTION FOR USE

Apply the dressing/compress on clean and dry skin using the tape to keep it in place. After use, please handle it according to medical waste. Do not apply tape too tight, this will influence the blood circulation and feel uncomfortable.

## CAUTION FOR USE

Do not apply the tape directly on the wound. External use only, keep away from children. If wound bleeds excessively or skin becomes irritated or infected remove the tape and consult a doctor.



## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat.

*Print and color may vary.*

## SPECIFICATIONS

<b>Product name</b>	BlueStar Non Woven Tape
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I
<b>Material</b>	Hot melt glue and PP non-woven
<b>Weight</b>	≥50 g/m <sup>2</sup> (per square meter with glue)
<b>Size</b>	2.5 cm x 5 meter
<b>Peel strenght</b>	≥2.0N/cm
<b>Holding adhesiveness</b>	≤2.5 mm
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2012

# BLUESTAR TRIANGULAR BANDAGE

## DESCRIPTION

To immobilise and keep splints in place. Use the bandage as a sling to support an injured, sprained or broken arm.

## DIRECTION FOR USE

Unfold bandage and attach around the neck as shown.

## CAUTION FOR USE

Product to be used once only. Use before the expiry date on the individual item. Check the integrity of the individual item before use.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat.

## SPECIFICATIONS



*Print and color may vary.*

<b>Product name</b>	BlueStar Triangular Bandage
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I
<b>Material</b>	Non-woven
<b>Weight</b>	38 gsm
<b>Size</b>	96 x 96 x 136 cm
<b>Shelf life</b>	5 years

# BLUESTAR STERILE WOUND DRESSING

## DESCRIPTION

Bandage with compress for protecting a clean and dry wound.

## DIRECTION FOR USE

Cleaning and drying the wound and the skin around the wound. Apply the compress on the wound and wrap the bandage around. Finish by tying the two ends together.

## CAUTION FOR USE

Replace the dressing on a daily basis, or more often if necessary. Single use only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet. Must not get wet.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat. Keep away from light and heat (store below 30 °C).



Print and color may vary.

## SPECIFICATIONS

<b>Product name</b>	BlueStar Sterile Wound Dressing
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I sterile R
<b>Material</b>	Bandage: 150D high stretch yarn polyester for radial direction, 32S polyester yarn Compress: 70% viscose, 30% polyester; PE film Package: White plastic
<b>Weight</b>	Bandage: 30 gsm (per square meter) Compress: 260 gsm (per square meter)
<b>Size</b>	Bandage: 8 cm x 2.5 meter Compress: 17 x 17 cm
<b>Shelf life</b>	5 years
<b>Sterile method</b>	Radiosterilization
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

# BLUESTAR AQR PLASTER

## DESCRIPTION

For use on minor cuts and abrasions. Protects minor wounds.

## FEATURES

- Water resistance
- Good air permeability

## DIRECTION FOR USE

Clean wound and surrounding area and dry skin before applying the plaster. Remove plaster from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25 °C.



*Print and color may vary.*

## CAUTION FOR USE

Use immediately after opening protective wrapper. Do not use with major wounds. Wrapper contains natural rubber latex. If wound bleeds excessively or skin becomes irritated or infected remove plaster and consult a doctor immediately. If you are diabetic consult your doctor before use.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat (store below 25 °C)

## SPECIFICATIONS

<b>Product name</b>	BlueStar AQR Plaster
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I sterile
<b>Material</b>	Absorbent pad: 70% viscose and 30% polyester Plaster film: PEVA
<b>Weight</b>	Absorbent pad: 130 gsm (per unit area)
<b>Size</b>	Type 1: 72 x 19 mm + Type 2: 72 x 25 mm Type 3: 56 x 19 mm + Type 4: 60 x 60 mm.
<b>Bioburden control</b>	Total bacteria <100 CFU/g Yeast/mouldiness ≤20 CFU/g Pathogenic: 0.
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

# BLUESTAR SKIN CLEANSING SWAB

## DESCRIPTION

For cleaning the skin and area around minor wounds and abrasion.

## FEATURES

- Saturated with 70% isopropyl alcohol

## DIRECTION FOR USE

Wipe carefully around the area to be cleaned.

## CAUTION FOR USE

For external use only. Avoid any contact with the eyes, mucosa, genitals and injured skin. Product to be used once only. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, contact a doctor immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the wipe in the environment.

## STORAGE

Should be stored at room temperature. For best results of product performance. Keep away from light and heat.

## SPECIFICATIONS

<b>Product name</b>	BlueStar Skin Cleansing Swab
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I sterile
<b>Material</b>	Swab: Non-woven Packing: Aluminium foil
<b>Weight</b>	Wet strength paper: 43 gsm (per square meter)
<b>Size</b>	Wet strength paper (unfolded): 3 x 6 cm Wet strength paper (folded): 3 x 3 cm Packing: 5 x 5 cm
<b>Sterile method</b>	R
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016



Print and color may vary.

# BLUESTAR DRY SWAB

## DESCRIPTION

For cleaning and drying the wound and the skin around the wound. Protecting a clean and dry wound.

## FEATURES

- Sterile

## DIRECTION FOR USE

Open the package and wipe gently in the area to be cleaned/dried. Or place the swab on the area that needs protection.

## CAUTION FOR USE

Product to be used once only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat.



*Print and color may vary.*

## SPECIFICATIONS

<b>Product name</b>	BlueStar Dry Swab
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I sterile
<b>Material</b>	Compress: Absorbing materials – 70% viscose, 30% polyester Packing: Paper film bag
<b>Weight</b>	30 gsm
<b>Size</b>	5 x 5 cm
<b>Sterile method</b>	EO
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016

# BLUESTAR ELASTIC BANDAGE

## DESCRIPTION

To hold compresses or dressings in place or for support on minor sprains.

## DIRECTION FOR USE

Wrap the bandage around the area the area to be supported, but do not apply excessive pressure. Trap the end under the lower layers or use a safety pin.

## CAUTION FOR USE

Never apply the bandage directly onto injured skin. Avoid pressing too hard. In case of discomfort, release the bandage to reduce the pressure. Do not wear the bandage overnight. Store in a cool, dry place.

## STORAGE

Should be stored in a cool dry environment for best results of product performance. Keep away from light and heat.

## SPECIFICATIONS

<b>Product name</b>	BlueStar Elastic Bandage
<b>Art.No.</b>	None (only by ordering refill)
<b>CE Class</b>	I
<b>Material</b>	Polyester
<b>Size</b>	8 cm x 4 meter (stretched length)
<b>Shelf life</b>	5 years
<b>Certificates TÜV SUD</b>	EN ISO 13485:2016



*Print and color may vary.*

## 1030200 WATERJEL (ONLY IN 1013100)

### DESCRIPTION

A small square-shaped dressing designed to fit the palm of the hand. Also good for the back of the hand, near the knuckles. Helps relieve pain, cool burns and protect against contamination. Sterile and foil packed.

### FEATURES

- Sterile
- Gel soaked
- Medical-grade non-woven material
- Absorbs temperatures in excess of 2000 °F

### DIRECTION FOR USE

Stop the burning process immediately. Cool the burn, don't overcool the victim. Provide pain relief. Cover and protect the burn area against contamination.

### STORAGE

Shouldn't be stored at temperatures in excess of 130 °F, but it will not harm the dressing.

### SPECIFICATIONS

<b>Product name</b>	Waterjel
<b>Art.No.</b>	1030200
<b>Material</b>	Medical-grade non-woven material
<b>Size</b>	10.1 x 10.1 cm
<b>Shelf life</b>	5 years



*Print and color may vary.*